

Demonstration case

Fully self-contained to realize disinfection programs

Superior maneuverability with its wheels and handle

Designed to carry Phileas®25



DIMENSIONS :

L = 630 mm

l = 500 mm

h = 300 mm

WEIGHT : 8 kg

The PHILEAS[®] case contains :

- The Phileas[®]25 with its cover :

- Power : 30 W
- Flow rate : 500 ml/h
- Volume of treatment : 3 m³ to 40 m³
- Volume of the container : 500 ml



- One 500 ml container

- One adapter and one power cable

(AC/DC Médical 150 Watts 24V6, 25A, Class II)



- One document pouch containing :



1. User manual
2. Simplified manual
3. Guarantee certificate
4. Safety instructions and Precautions for use
5. Quality checklist

- Tablet with its power cable. MyPhileas[®] application is directly loaded.

This application allows the remote piloting and guarantees the traceability of the realised disinfections.



Test Phileas[®]25

To check the operation of Phileas[®]25, we recommend you to realize a water test :

1. Connect your unit to an electrical installation which complies with safety standards. The cable supplied is standard and meets EU, TUV, ANSI / AAMI CSA, UL and Denan (PSE) standards.
2. Fill the container with tap water.
3. Start Phileas[®]25
4. Follow the procedure outlined on page 14 of the user manual.